




Fluydo NC

NON-COMPLIANT PTCA RX BALLOON DILATATION CATHETER

I N T E R V E N T I O N A L C A R D I O L O G Y

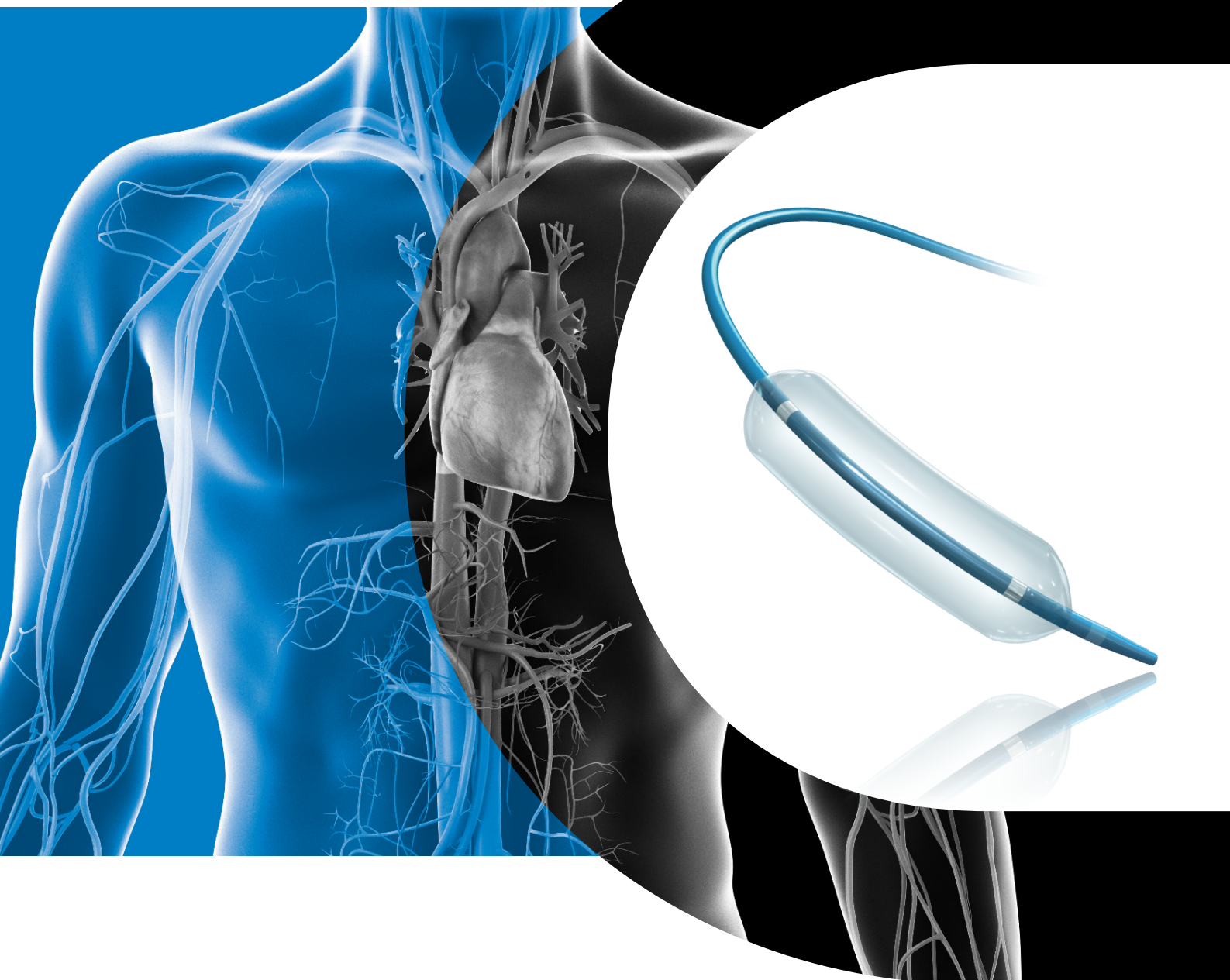


VERSATILE NON-COMPLIANT BALLOON
WITH SUPERB CROSSABILITY

PERFECT COMBINATION OF HIGH RBP
WITH ULTRA LOW COMPLIANCE

6F KISSING BALLOON TECHNIQUE
FULLY COMPATIBLE*

TECHNICAL DATA

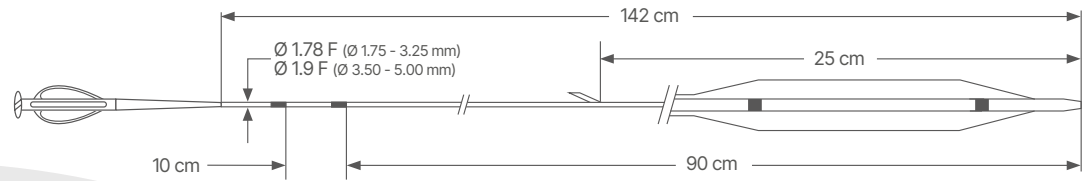

CATHETER SPECIFICATIONS	
Category	Rapid Exchange Balloon catheter
Guiding catheter compatibility	5F
Guidewire compatibility	0.014"
Usable catheter length	142 cm
Proximal shaft	Stainless steel AISI 304, low friction hypotube with Teflon coating
Distal shaft	Mix Polyamide/PEBA with hydrophilic coating
Tip material	PEBA blend
Entry profile	0.017"

BALLOON SPECIFICATIONS	
Category	Non-compliant
Material	Polyamide
X-ray visibility	Two Platinum markers
Nominal Pressure	12 atm
Rated Burst Pressure	22 atm (1.75 mm to 4.00 mm) 20 atm (4.50 mm) 19 atm (5.00 mm)
Average Burst Pressure	28 atm (1.75 mm to 4.00 mm) 26 atm (4.50 mm to 5.00 mm)

COMPLIANCE CHART

■ Nominal Pressure ■ Rated Burst Pressure

Pressure (atm) \ Ø (mm)	1.75	2.00	2.25	2.50	2.75	3.00	3.25	3.50	3.75	4.00	4.50	5.00
12	1.75	2.00	2.25	2.50	2.75	3.00	3.25	3.50	3.75	4.00	4.50	5.00
14	1.77	2.03	2.28	2.53	2.79	3.04	3.29	3.55	3.80	4.05	4.56	5.07
16	1.80	2.05	2.31	2.56	2.82	3.08	3.33	3.59	3.85	4.10	4.62	5.13
18	1.82	2.08	2.34	2.60	2.86	3.11	3.38	3.64	3.89	4.16	4.68	5.20
19	1.83	2.09	2.35	2.61	2.88	3.13	3.40	3.66	3.92	4.18	4.71	5.23
20	1.85	2.10	2.37	2.63	2.89	3.15	3.42	3.68	3.94	4.21	4.74	
22	1.87	2.13	2.40	2.66	2.93	3.19	3.46	3.73	3.99	4.26		

ORDER INFORMATION

Ø (mm) \ L (mm)	6	8	12	15	20	30
1.75	ICNC17506	ICNC17508	ICNC17512	ICNC17515	ICNC17520	
2.00	ICNC20006	ICNC20008	ICNC20012	ICNC20015	ICNC20020	
2.25	ICNC22506	ICNC22508	ICNC22512	ICNC22515	ICNC22520	
2.50	ICNC25006	ICNC25008	ICNC25012	ICNC25015	ICNC25020	ICNC25030
2.75	ICNC27506	ICNC27508	ICNC27512	ICNC27515	ICNC27520	
3.00	ICNC30006	ICNC30008	ICNC30012	ICNC30015	ICNC30020	ICNC30030
3.25	ICNC32506	ICNC32508	ICNC32512	ICNC32515	ICNC32520	
3.50	ICNC35006	ICNC35008	ICNC35012	ICNC35015	ICNC35020	ICNC35030
3.75	ICNC37506	ICNC37508	ICNC37512	ICNC37515	ICNC37520	
4.00	ICNC40006	ICNC40008	ICNC40012	ICNC40015	ICNC40020	
4.50		ICNC45008	ICNC45012	ICNC45015	ICNC45020	
5.00		ICNC50008	ICNC50012	ICNC50015	ICNC50020	

*6F kissing balloon technique fully compatible: any two balloons with at least one with diameter ≤ 3,25 mm



CID S.p.A.

Manufacturer:

Viale Certosa 218 - 20156 Milano (MI) Italy

Manufacturing Facility:

Via Crescentino snc - 13040 Saluggia (VC) Italy

www.cidvascular.com

Please contact your local sales representative for product availability. The images shown within this document are of a purely illustrative nature and are not intended to depict reality. Please consult product labels and package inserts for indications, contraindications, hazards, warnings, cautions and instruction for use.